

Tamarack® Brim Sheath

with **GlideWear™** technology



Low-friction GlideWear™ fabric **reduces friction and shear**, protecting skin at risk for painful rubbing and breakdown.

Features & Benefits

- Featuring Knit-Rite Design for Above-Knee Amputees
- Get **lasting protection from painful friction** where your socket brim comes into contact with your skin
- **Comfortable, breathable fabric** protects skin from painful pinching and rubbing
- Use with most socket styles and liner systems, including roll-on liner, pin suspension, or seal-in liner

Suggested billing: L8410 Prosthetic Sheath, AK each

AVAILABLE IN FOUR SIZES

PART NO.	SIZE	SOCKET CIRCUMFERENCE (IN, CM)	
GW-KRS-S	Small (S/P)	14" - 18"	35 - 45 cm
GW-KRS-M	Medium (M)	18" - 22"	46 - 55 cm
GW-KRS-L	Large (L/G)	22" - 30"	56 - 76 cm
GW-KRS-XL	Extra Large (XL/XG)	30" - 40"	77 - 101 cm

Note: Measure the circumference of the **widest** part of your socket.



CARE INSTRUCTIONS



Wash with mild detergent

85% Nylon, 15% Spandex
DOES NOT CONTAIN LATEX

See reverse side for installation instructions.

Tamarack Habilitation Technologies, Inc.
1670-94th Lane NE, Blaine, MN 55449
Tel: 866.795.0057
E-mail: info@tamarackhti.com



QNET BV
Kantstraat 19
NL-5076 NP Haaren
The Netherlands




ISO13485: 2016 Registered QMS Rev. 10/19

© 2019 Tamarack Habilitation Technologies, Inc.
GlideWear is a registered trademark of MIPS, manufactured in the USA by Tamarack HTI.
Patented US8646459 - US9241853 - GB2478206 - CA2738162 - Other Patents Pending.
All Rights Reserved.

Installation Instructions

Tip: watch the installation video at tamarackhti.com

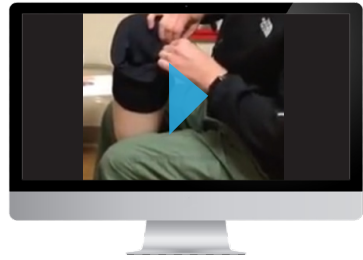

1

Step 1: Make sure the GlideWear label and silicone dots are on the outside.



2

Step 2: Hold the Brim Sheath by the top band (with the silicone dots on the outside), then insert the bottom part of the Brim Sheath into your socket.



3

Step 3: Flip the top band over the top of your brim (so that the silicone dots are touching the outside of your brim), while keeping the rest of the Brim Sheath inside your socket.



4

Step 4: Pull band down until seam is just below the edge of your brim on the outside.

Step 5: Don your prosthesis as you normally do.

Note: The Brim Sheath may not be effective for all socket shapes and designs. Consult your prosthetist if you are experiencing problems with the fit or function of your socket.



5



Questions? We're here to help. Call us at 866.795.0057 or email info@tamarackhti.com