



Tamarack
HABILITATION TECHNOLOGIES, INC.


Tamarack Flexure Joint™

Hand Punch Tool

The Hand Punch tool has a “nose” designed to fit precisely into cavities molded in plastic orthoses for Tamarack Flexure Joints™. It allows you to punch perfectly located, burr-free holes for the Tamarack Flexure Joint™ mounting hardware. Large, Medium and Pediatric sizes are available.

After the cut separating the foot and calf sections, place the nose of the punch into the cavity; push it gently against the end of the cavity. Now punch the hole. The guide pin automatically aligns the punch frame during the punching process.

The Tamarack Hand Punch tool may also be used for general hole punching in plastic, leather, rubber, etc. *Refer to thickness recommendations below for additional details.*

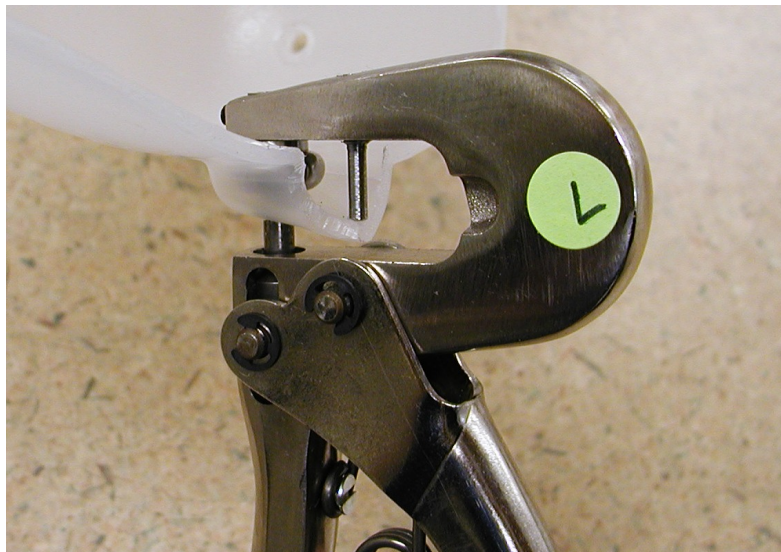

Attention: The Tamarack Hand Punch was designed to punch through polypropylene material no greater than 3/16” (5mm) thick. Attempting to punch harder or thicker materials may result in damage to the device or tool.

Manufactured in the USA by
Tamarack Habilitation Technologies
www.tamarackhti.com
info@tamarackhti.com
(866) 795-0057 toll-free

Distributed Worldwide by
Becker Orthopedic
www.beckerorthopedic.com
mail@beckerorthopedic.net
(800) 521-2192 toll-free