



For more information about our products, visit

www.tamarackhti.com

Email: info@tamarackhti.com

Call: (763) 795-0057

Toll-Free: (866) 795-0057

Scan for distributor and ordering information!



Tamarack Habilitation Technologies, Inc.
1670-94th Lane NE, Blaine, MN 55449
Tel: 866.795.0057
E-mail: info@tamarackhti.com



ISO13485: 2016 Registered QMS Rev. 7/23

© 2023 Tamarack Habilitation Technologies, Inc.
GlideWear is a registered trademark of MIPS, manufactured in the USA by Tamarack HTI | Patented US8646459 - US9241853 - GB2478206 - CA2738162 - Other Patents Pending | SmartKnit is a registered trademark of KnitRite | All Rights Reserved

Tamarack® AK Brim Sheath

with **GlideWear™**



GW-KRS

Targeted protection along the
AK socket brim.



Suggested Billing:
L8410 Prosthetic Sheath, AK (each)

Tamarack® AK Brim Sheath

Low-friction GlideWear™ fabric **reduces friction and shear**, protecting skin at risk for painful rubbing and breakdown.

GW-KRS Features

- Featuring Knit-Rite design for above-knee amputees
- **Lasting protection from painful friction** where the socket brim comes into contact with skin
- **Comfortable, breathable fabric** protects skin from pinching & rubbing
- Designed for most socket styles and liner systems (including roll-on liner, pin suspension, or seal-in liner)
- **Durable, machine washable, & latex-free**

PART NO.	SIZE	SOCKET CIRCUMFERENCE*
GW-KRS-S	Small (S/P)	14-18 in 35-45 cm
GW-KRS-M	Medium (M)	18-22 in 46-55 cm
GW-KRS-L	Large (L/G)	22-30 in 56-76 cm
GW-KRS-XL	XLarge (XL/XG)	30-40 in 77-101 cm

* Measure the circumference of the **widest** part of your socket.

CARE INSTRUCTIONS

Wash with mild detergent.



MATERIALS

85% Nylon, 15% Spandex
DOES NOT CONTAIN LATEX

Installation Instructions

1. Make sure the GlideWear label and silicone dots are facing out.
2. Hold the brim sheath by the top band (with the silicone dots facing out) and insert the bottom part of the brim sheath into your socket.
3. Fold the top band over the top edge of your brim - the silicone dots will now face and grip the outside of your socket - while keeping the rest of the brim sheath inside of the socket.
4. Pull the band down until the center seam is just below the outside edge of your brim - the GlideWear™ portion of the brim sheath will now be protecting the edge of the brim and extend inside your socket.
5. Don your prosthesis.



Scan for our Brim Sheath Installation Video!



NOTE: The brim sheath may not be effective for all socket shapes and designs. Consult your prosthetist if you experience problems with the fit or function of your socket.