

GW-SIT Cushion Cover Installation

GlideWear® reduces friction and shear under the bony prominences of your pelvis; complementing traditional pressure reduction methods provided by your wheelchair cushion.

SIZE SELECTION

The GlideWear® cushion cover can be used on most skin protection and positioning cushions. When properly fitted, the cover should be wrinkle-free without being stretched too tightly in any direction.

Covers are sized for standard height cushions - please contact Tamarack to inquire about custom solutions.

Small (S/P)	Fits 14 in. x 14 in. to 16 in. x 16 in. cushions (35 cm x 35 cm to 40 cm x 40 cm)
Medium (M)	Fits 16 in. x 16 in. to 18 in. x 18 in. cushions (40 cm x 40 cm to 45 cm x 45 cm)
Large (L/G)	Fits 18 in. x 18 in. to 20 in. x 20 in. cushions (45 cm x 45 cm to 50 cm x 50 cm)

INSTALLATION

- 1 Wrap the GlideWear® cushion cover over your cushion, making sure the GlideWear® dual-layer fabric segment is positioned over the rear of the cushion.



- 2 If cushion uses hook and loop tape to attach to the wheelchair, use sharp scissors to cut two to four 2" (5 cm) holes in the bottom panel to allow fastening into the wheelchair.



- 3 Place the cushion into the wheelchair in the proper orientation. Ensure the GlideWear® dual layer fabric segment is positioned nearest the wheelchair backrest.



PRECAUTIONS

- Ensure the dual layer fabric is properly positioned at the rear of the cushion beneath your buttocks
- Use caution when first using the GlideWear® cushion cover; observing for any unintentional sliding or posture change while sitting. If any unintended stability changes occur, discontinue use and consult with your seating specialist for assistance.
- Do not use GlideWear® turned inside out.
- Avoid contact with hook & loop (Velcro® type) fasteners, which will snag the fabric.
- To receive maximum friction and shear-reducing benefits, it is recommended to not place any additional covers or materials between the GlideWear® cushion cover and users clothing.

CARE INSTRUCTIONS

Wash with mild detergent.



MATERIAL: 85% Nylon 15% Spandex

DOES NOT CONTAIN LATEX

DISPOSAL: Recycle per federal, state, or local regulations.

Contact Us

For more information, visit:

www.tamarackhti.com

or email: info@tamarackhti.com

