# ShearBan<sup>®</sup>

by Tamarack<sup>®</sup>

TR | JA

Single & 5-Sheet Packs  
8 in. x 12 in.  
(20 cm x 30 cm)

Rivet Cover Patches  
3/4 in. and 15/16 in. round  
(2 cm. and 2.5 cm. round)

## Installation Instructions

- 1) Clean application surface as necessary to remove dirt and moisture.
- 2) Cut desired shape/size of ShearBan with sharp scissors, ensuring the ShearBan patch will extend beyond the area of required coverage.
- 3) Round all corners of the ShearBan patch to prevent peeling after installation.
- 4) Apply ShearBan directly to the footwear, insole, orthosis or prosthesis – not against the skin. Press firmly around entire patch surface to ensure adhesion.

For more info, visit  
[www.ShearBan.com](http://www.ShearBan.com)

**Tamarack<sup>®</sup>**  
HABILITATION TECHNOLOGIES, INC.

 Tamarack Habilitation Technologies  
1670-94th Lane NE, Minneapolis, MN 55449 USA  
Tel: 1-763-795-0057  
E-mail: [info@tamarackhti.com](mailto:info@tamarackhti.com)

## User Notice:

Tamarack does not advocate ShearBan as a substitute for proper contouring of supportive devices and is not responsible for undesirable results due to improper friction management practices. ShearBan, an interface material, reduces the friction and shear that can cause tissue trauma in orthotic and prosthetic applications. The self-adhesive back is to be applied to body support or containment surfaces, not to the human skin. ShearBan's low friction surface is comprised of a PTFE film which is intended to bear against an item of fabric clothing or directly against healthy, intact skin. It should not be used to directly cover or be placed against open wounds or abraded skin without an additional wound care dressing between ShearBan and the skin. For best results, store unused ShearBan in the bag provided.

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